



LEGION RK HK Finned Wedges Quick Reference Guide

LEGION RK HK Finned Full Tibial Wedges are a new addition to the LEGION Revision portfolio and are designed to provide additional rotational stability of a knee construct that contains a full tibial wedge by extending the existing tibial tray fin profile distally. The new wedges are hand and size specific with the addition of fins. As with the existing full wedges, the finned wedges taper distally one size smaller than the baseplate used with every 10mm of thickness, thus, the fins are smaller than those on the baseplate.

 **smith&nephew**

LEGION[◊] RK

Revision Knee System

LEGION[◊] HK

Hinge Knee System

Supporting healthcare professionals

LEGION[◊] RK HK Finned Tibial Wedge Quick Reference Guide

If no full wedge trial is assembled to the tibial trial tray prior to using the fin punch, simply punch for the fins using one size smaller fin punch than the tibial baseplate size (Table 1).

If a full wedge trial is assembled to the tibial trial tray prior to using the fin punch, then the tibial wedge will need to be removed before fin punching.

In these cases follow the steps below:

1. Mark the rotation of the tibial baseplate on the proximal tibia
2. Remove the tibial trial construct
3. Remove the wedge trials from the tibial baseplate
4. Put the tibial baseplate back in place and align with the rotation mark from step one
5. Fin punch with one size smaller fin punch than the tibial baseplate used (Table 1)



For detailed product information, including indications for use, contraindications, precautions and warnings, please consult the product's applicable Instructions for Use (IFU) prior to use.

Table 1. LEGION RK HK Finned Wedge Fin Punch Size

Left Finned Wedges

Catalog Item	Description	Fin Punch Size
7142-6134	LEGION Finned Tibial Wedge Sz2 10mm Left	Sizes 1-2
7142-6135	LEGION Finned Tibial Wedge Sz2 15mm Left	Sizes 1-2
7142-6136	LEGION Finned Tibial Wedge Sz2 20mm Left	Sizes 1-2
7142-6137	LEGION Finned Tibial Wedge Sz3 10mm Left	Sizes 1-2
7142-6138	LEGION Finned Tibial Wedge Sz3 15mm Left	Sizes 1-2
7142-6139	LEGION Finned Tibial Wedge Sz3 20mm Left	Sizes 1-2
7142-6141	LEGION Finned Tibial Wedge Sz4 10mm Left	Sizes 3-4
7142-6142	LEGION Finned Tibial Wedge Sz4 15mm Left	Sizes 3-4
7142-6143	LEGION Finned Tibial Wedge Sz4 20mm Left	Sizes 3-4
7142-6144	LEGION Finned Tibial Wedge Sz5 10mm Left	Sizes 3-4
7142-6145	LEGION Finned Tibial Wedge Sz5 15mm Left	Sizes 3-4
7142-6146	LEGION Finned Tibial Wedge Sz5 20mm Left	Sizes 3-4
7142-6147	LEGION Finned Tibial Wedge Sz6 10mm Left	Sizes 5-6
7142-6148	LEGION Finned Tibial Wedge Sz6 15mm Left	Sizes 5-6
7142-6149	LEGION Finned Tibial Wedge Sz6 20mm Left	Sizes 5-6
7142-6151	LEGION Finned Tibial Wedge Sz7 10mm Left	Sizes 5-6
7142-6152	LEGION Finned Tibial Wedge Sz7 15mm Left	Sizes 5-6
7142-6153	LEGION Finned Tibial Wedge Sz7 20mm Left	Sizes 5-6
7142-6154	LEGION Finned Tibial Wedge Sz8 10mm Left	Sizes 7-8
7142-6155	LEGION Finned Tibial Wedge Sz8 15mm Left	Sizes 7-8
7142-6156	LEGION Finned Tibial Wedge Sz8 20mm Left	Sizes 7-8

Right Finned Wedges

Catalog Item	Description	Fin Punch Size
7142-6161	LEGION Finned Tibial Wedge Sz2 10mm Right	Sizes 1-2
7142-6162	LEGION Finned Tibial Wedge Sz2 15mm Right	Sizes 1-2
7142-6163	LEGION Finned Tibial Wedge Sz2 20mm Right	Sizes 1-2
7142-6164	LEGION Finned Tibial Wedge Sz3 10mm Right	Sizes 1-2
7142-6165	LEGION Finned Tibial Wedge Sz3 15mm Right	Sizes 1-2
7142-6166	LEGION Finned Tibial Wedge Sz3 20mm Right	Sizes 1-2
7142-6167	LEGION Finned Tibial Wedge Sz4 10mm Right	Sizes 3-4
7142-6168	LEGION Finned Tibial Wedge Sz4 15mm Right	Sizes 3-4
7142-6169	LEGION Finned Tibial Wedge Sz4 20mm Right	Sizes 3-4
7142-6171	LEGION Finned Tibial Wedge Sz5 10mm Right	Sizes 3-4
7142-6172	LEGION Finned Tibial Wedge Sz5 15mm Right	Sizes 3-4
7142-6173	LEGION Finned Tibial Wedge Sz5 20mm Right	Sizes 3-4
7142-6174	LEGION Finned Tibial Wedge Sz6 10mm Right	Sizes 5-6
7142-6175	LEGION Finned Tibial Wedge Sz6 15mm Right	Sizes 5-6
7142-6176	LEGION Finned Tibial Wedge Sz6 20mm Right	Sizes 5-6
7142-6177	LEGION Finned Tibial Wedge Sz7 10mm Right	Sizes 5-6
7142-6178	LEGION Finned Tibial Wedge Sz7 15mm Right	Sizes 5-6
7142-6179	LEGION Finned Tibial Wedge Sz7 20mm Right	Sizes 5-6
7142-6181	LEGION Finned Tibial Wedge Sz8 10mm Right	Sizes 7-8
7142-6182	LEGION Finned Tibial Wedge Sz8 15mm Right	Sizes 7-8
7142-6183	LEGION Finned Tibial Wedge Sz8 20mm Right	Sizes 7-8

Smith & Nephew, Inc.
1450 Brooks Road
Memphis, TN 38116
USA

Telephone: 1-901-396-2121
Information: 1-800-821-5700
Orders/Inquiries: 1-800-238-7538

www.smith-nephew.com

™Trademark of Smith & Nephew
©2018 Smith & Nephew.
14855 VI 05/18

Supporting healthcare professionals for over 150 years