


1. Expose the bone at the desired insertion location for the implant. Attach the appropriate drill bit to a standard orthopedic drill. Be sure to chuck the drill no farther than the flats on the proximal end of the drill.



2. Position the tip of the Drill Guide with Obturator against the bone surface at the desired position, then remove the Obturator. The crescent laser mark indicates the direction of the curve.

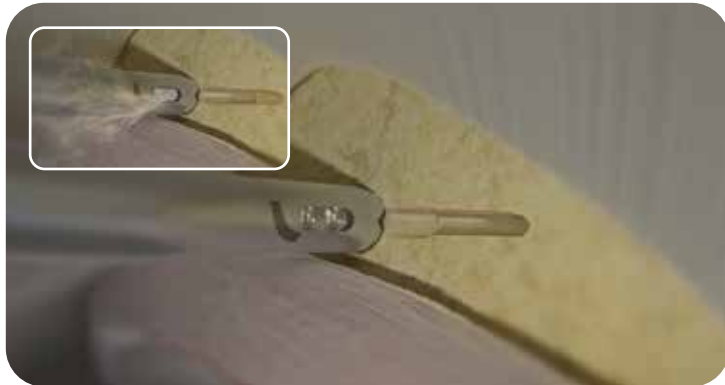

3. Insert the appropriate drill into the drill guide. For dense bone, it may be required to cycle the drill in and out of the implantation site to ensure complete removal of bone. Next, remove the drill.



4. Insert the preloaded suture anchor into the proximal end of the drill guide. While maintaining distal pressure on the insertion device, push and/or tap on the proximal end of the insertion device to insert the anchor. Be sure that the anchor inserter is fully engaged with the drill guide prior to anchor deployment.



5. To disengage the anchor, hold the handle securely in the palm of the hand and pull back on the finger grips using two fingers to retract the suture tensioning mechanism.



6. Pass sutures through tissue and complete repair using desired technique. (Repeat steps 2 through 6 to complete repair).

Ordering information

MICRORAPTOR® REGENESORB® Suture Anchors

Reference #	Description
72204983	MICRORAPTOR REGENESORB Suture Anchor with ONE ULTRABRAID® #1 Suture (Blue)
72204984	MICRORAPTOR REGENESORB Suture Anchor with ONE ULTRABRAID #1 Suture (Blue COBRAID)

MICRORAPTOR REGENESORB Drill Guides, Drills and Obturators

Reference #	Description
72204988	MICRORAPTOR Drill, 1.6mm
72205267	MICRORAPTOR Drill, Hard Bone, 1.8mm
72204991	MICRORAPTOR Drill Guide, Crown Tip
72204992	MICRORAPTOR Drill Guide, Spike Tip
72204993	MICRORAPTOR Drill Guide, Crown Tip, Curved
72204995	MICRORAPTOR Drill Guide, Fishmouth Tip
72204999	MICRORAPTOR Obturator, Blunt Tip, Cannulated
72205000	MICRORAPTOR Obturator, Blunt Tip, Cannulated
72205001	MICRORAPTOR Obturator, Trocar Tip

All photos and diagrams are for illustrative purposes only. Products not to scale.

For more information on the application of any products discussed in the presentation, as well as indications for use, contraindications, and product safety information, please consult the Instructions for Use (IFU) for sure product. To order these instruments, call **+1 800 343 5717** in the U.S. or contact an authorized Smith+Nephew representative.

Learn more at [smith-nephew.com](https://www.smith-nephew.com)